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**LATEX
FREE**

**POWDER
FREE**

 Resistive exercise products play a vital role in home and clinical rehabilitation regimens and strengthening programs prescribed by health care professionals. Today, progressive resistance exercise products are used for rehabbing muscle and joint injuries, work hardening programs, and aerobic and aquatic exercise programs.

REP Band® Products can play an integral part in these programs to strengthen muscles and increase endurance.

What differentiates REP Band® Resistive Exercise Products from other resistive exercise products is that they are made of non-latex material and are powder and odor-free. A quality resistive exercise product that will not trigger latex allergies or leave a dusty powder on your clothes after exercising.

NOTE: The instructions in this guide are just indicative and not intended to be a prescription for exercises.

Before starting any exercise program, it is recommended to consult a specialist who is watching you.

The REP Band products are not toys. Children must be supervised at all times.

Safety precautions: If not used properly, the REP Band products can cause injury.

Never use REP Band products in a way to make the film be shot in the head and eyes.

Before use, always consider the REP Band products for defects or damage.

Without using the REP Band products when flying small cuts or holes that can cause tearing or breaking of the film during exercise.

Replace the damaged product with a new one. Protect REP Band products from sharp objects to avoid accidental punctures and tears.

Practical Tips : For exercises you need to tie the exercises movies REP Band in loop, use a half bow or knot.

Always make sure that the knot or half bow is secure before starting the exercise. In use you can wear long socks to prevent the leg pulled hair from the movie. Do not tighten the joints when you exercise.

Always hold the elbows and your knees slightly bent. Breathe normally during exercise do not hold your breath!

Maintain proper body alignment when performing exercises in a standing position. Keep your shoulders back and your body erect.

Storage and Care of Rep Band : Keep REP Band products in a cool (21 ° C), dark place. When using the movies in pools containing chlorine, thoroughly rinse the film after each use and let it air dry.

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